**S-400-B**

**SELLADORA  
SEALING MACHINE**

**4100047**

## Indice

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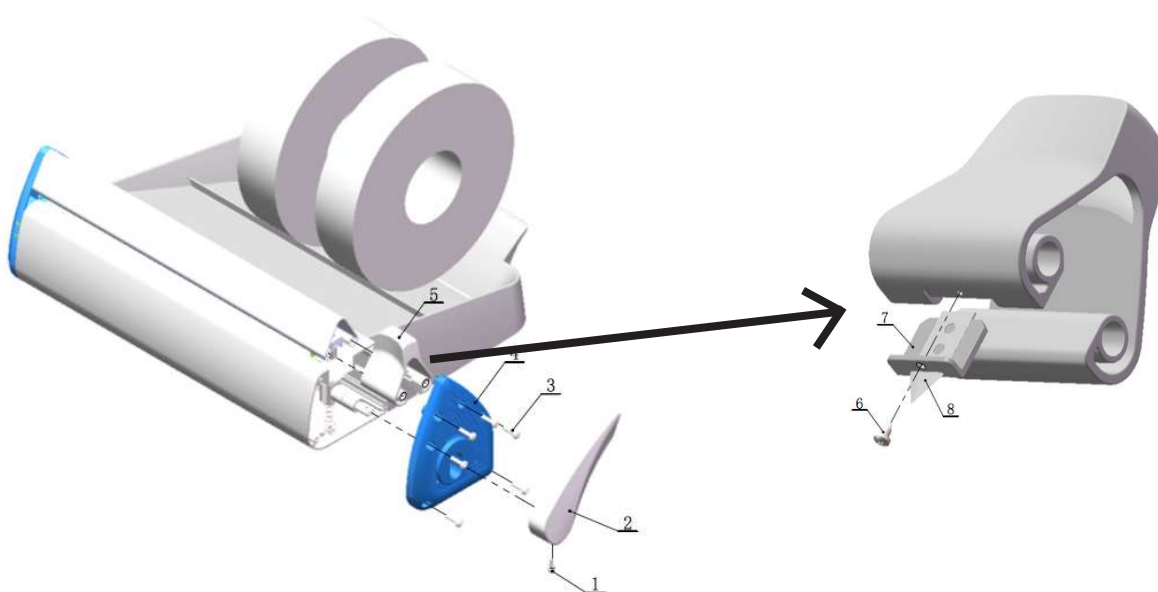
## 1. General information

- Handle the parcel with care. Unpack and check that the contents coincide with the packing-list. If any part is damaged or missing, please advise the distributors immediately.
- Do not install or use the equipment without reading this handbook first. This instruction manual must be attached to the equipment and must be available for all users.
- If you have any doubts or enquiries, contact with J.P. SELECTA's technical service.
- Do not use the equipment in explosive, flammable or corrosive atmospheres.

## 2. Packing list

Sealing machine	<b>4100047</b>
Power cable	
Sterilization paper roll (sample)	1 meter
Blade	
Instructions manual	80354

## 3. Technical specifications

Model:	<b>4100047</b>
Input power:	230V ~, 50 / 60Hz
Rated power:	100W
Sealing temperature:	200°C ± 10°C
Max. sealing temperature:	300mm
Sealing width:	12mm
Net weight:	4.8Kgs
Gross weight:	7Kgs
Package size:	50x40x21cm

## 4. Safety information

**Hazard  
risk**

**Electric  
risk**

**Risk of high  
temperature contact**

### 4.1 Risks the operator is subjected to

- Risk of burns
- Electrical risk

### 4.2 Staff qualification

This equipment may only be used by properly trained personnel who have read and understood these instructions or have been properly trained in this equipment operation.

### 4.3 Use with hazardous substances

Do not treat flammable or explosive materials or those which chemically react with violence.

### 4.4. Maintenance

Unplug the equipment from the mains before removing the oven's cover to handle inside.

**The manipulation of the equipment internal electronic circuits by unauthorized personnel can cause irreparable damage. Take it to one of J.P.SELECTA's authorized technical services.**

It is recommended to clean the accessible parts of the oven with a cloth and a cleaner, or a similar solution. Do not use aggressive solvents.

### 4.5. Warranty

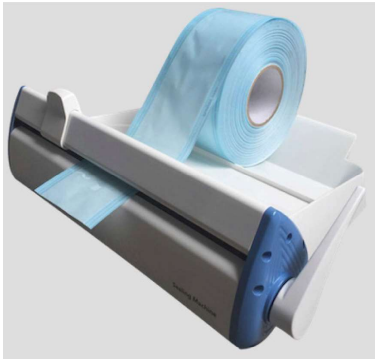
This product is guaranteed for one year. It does not cover damages caused by an incorrect use or causes beyond the control of J. P. SELECTA, s.a.u.

Any manipulation of the equipment by personnel not authorized by J.P. SELECTA, s.a.u. automatically cancels the guarantee.

## 5. Equipment description

Bag sealing machine S-400-B is been designed for use in dental clinics. The equipment will need to be stored in a bacterial box or packaged with the sterilization roll. The sealing machine is a very necessary and useful equipment to pack instrumental. It has a precise temperature control, which makes it a high performance equipment.

## 6. Installation

- Plug in the power cord and turn the power switch on.
- The green light will blink after turning the power switch on. Preheating will take about 3 minutes.
- When the green light stops blinking and stays on, the sealing machine is ready to seal.
- Insert the sterilization paper roll, be sure to insert it correctly (see image).
- Press the handle forwards or backwards for 3 seconds, the machine will produce a sound to indicate that the machine is on, the yellow light will remain on, which means that it works correctly. If you do not let the handle go for 6 seconds, the buzzer will sound steadily.

## 7. Maintenance

- To clean the machine, first turn off the machine and wipe with a soft cloth.
- To replace the blade:
  1. Unscrew the handle fixing screw with Allen wrench D3 (# 1)
  2. Unscrew the other 6 screws with the cross-head screwdriver (# 2, # 3 and the rest)
  3. Remove the side plastic (part # 4)
  4. Remove the blade assembly (# 5)
  5. Unscrew screw # 6 with a small cross-head screwdriver.
  6. Remove the blade plate (# 7)
  7. Replace the blade (# 8)
  8. Reassemble all parts.

Note: Make sure that the machine is cool when replacing the blade. Be careful with the blade, it is very sharp.