SEAL-COM 3

**SELLADORA DE BOLSAS AUTOMÁTICA
CON IMPRESORA
AUTOMATIC BAG SEALING MACHINE
WITH PRINTER**

4100049

Indice

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1. General information

- Handle the parcel with care. Unpack and check that the contents coincide with the packing-list. If any part is damaged or missing, please advise the distributors immediately.
- Do not install or use the equipment without reading this handbook first. This instruction manual must be attached to the equipment and must be available for all users.
- If you have any doubts or enquiries, contact with J.P. SELECTA's technical service.
- Do not use the equipment in explosive, flammable or corrosive atmospheres.

2. Packing list

Sealing machine	4100049	(1pcs)
Power cable 3G075 (1.8m long)		(1 pcs)
Spare Fuses 5A / 250V		(2 pcs)
Ink cartridge		(1 pcs)
Guide plate		(1 pcs)
Sterilization paper roll		(samples)
Instructions manual	80393	(1 pcs)
EC Conformity Declaration		(1 pcs)
Quick Guide of Operation (plastic wrapped)		(1 pcs)

3. Technical specifications

Model:	4100049
Paper movement speed:	10m x min
Sealing width:	75mm (narrow)/106 (normal) / 140 (width)
Sealing temperature:	60-210°C
Power supply:	220V +10% 50Hz
Nominal Power:	500 W
Fuse:	5A *2
Weight:	18Kgs
Storage temperature:	-40 to 55°C
Humidity:	Less than 90%
Atmospheric pressure:	50KPa - 106KPa
Dimensions (high / wide / deep)	220 x 560 x 260 mm

4. Safety information

**Hazard
risk**

**Electric
risk**

**Risk of high
temperature contact**

4.1 Risks the operator is subjected to

- Risk of burns
- Electrical risk

4.2 Staff qualification

This equipment may only be used by properly trained personnel who have read and understood these instructions or have been properly trained in this equipment operation.

4.3 Use with hazardous substances

Do not treat flammable or explosive materials or those which chemically react with violence.

4.4. Maintenance

Unplug the equipment from the mains before removing the oven's cover to handle inside.

The manipulation of the equipment internal electronic circuits by unauthorized personnel can cause irreparable damage. Take it to one of J.P.SELECTA's authorized technical services.

It is recommended to clean the accessible parts of the oven with a cloth and a cleaner, or a similar solution. Do not use aggressive solvents.

4.5. Warranty

This product is guaranteed for one year. It does not cover damages caused by an incorrect use or causes beyond the control of J. P. SELECTA, s.a.u.

Any manipulation of the equipment by personnel not authorized by J.P. SELECTA, s.a.u. automatically cancels the guarantee.

5. Equipment description

Bag automatic sealing machine **Seal-Com 3** with printer is a special equipment to seal plastic bags of thermoplastic composite packaging material.

The sealing machine is a very necessary and useful equipment to pack instrumental. It has a precise temperature control, which makes it a high performance equipment.

6. Features

Main features of **Seal-Com 3** are: simple operation, automatic temperature control, display TFT, small size, comfortable movement and reliable work. The **Seal-Com 3** sealing machine can set the sealing temperature according to the requirements.


Automatic sealing, stainless steel and painted housing, transmission system, heating system, temperature control system, automatic feed induction system and printing system.

7. Installation

Connect the equipment to the power outlet and turn on the switch (the normal start will have a normal start tone); The synchronous motor will produce automatic operation for 10 seconds, a welcome message will be displayed on the screen. After 10 seconds, it will automatically enter the main screen.

7.1 Main screen

Upon entering the main screen, the first cell in the upper right corner indicates the system time (for example, the sterilization date). Press and hold the box and the keyboard will appear.

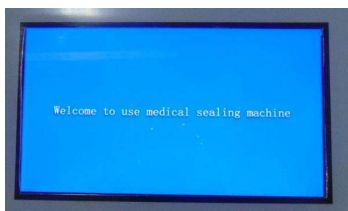A thermometer is also shown on the main screen, which indicates the temperature setting. 180°C is the normal set temperature. You can choose between two languages: English and Chinese (font or symbols). If you select font, there are three different types, divided into three possible widths: narrow, normal or wide.

On the main screen, the printer settings are also set. Press the printer icon to print the content.

7.2 Guide plate


To seal, adjust the guide plate and sealing position. To adjust it, turn the nut counterclockwise to move the guide plate outward, so that the seal is near the edge of the paper bag, or inward, so that the seal is far from the edge. The size is adjusted according to the type of seal. Once adjusted, turn the nut clockwise to fix the guide plate.

7.3 Tape

When the print is blurred, the tape should be replaced. Rotate the sealer, open the bottom cover, rotate the printhead, hold the spring clip, remove the tape, attach the new one. Replace the spring clamp, rotate the print head to the original position and close the bottom cover.

7.4 Replacement printer

Press configuration key on the main screen, enter password 1212 and change between default or alternative printer.

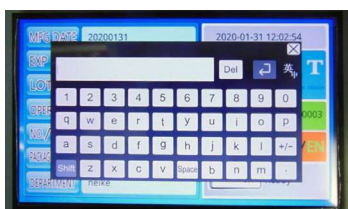8. Printer settings

1. Temperature settings:

Temperature can be adjusted to the needs of sealing. Factory settings are 180°C for sterilization bags and 120-130°C for plasma bags. You can set a new value and later on restore the factory values.

2. Print content settings and changes:

- When the printing function is not required, you can touch the printer icon on the screen to cancel printing. A red prohibition symbol will appear. If printing needs to be reactivated, press the same icon again.
- Touch the settings icon on the screen and enter the password "1212" to enter the printer setting options. You will be able to select between the internal printer or the replacement printer.

3. Traceability function:

Click on the Traceability symbol. You can search the operation logs through the printer screen or you can download this data through a USB pendrive and save it on a PC.

4. Auxiliary functions:

- You can set a "stand-by" time and temperature. After the inactivity state time, the machine will automatically return to the set temperature.
- Adjust the initial printing distance, select between 1 and 10mm.

5. Points that need attention:

- The opaque side of the paper/plastic bags should be placed down, and the transparent side up. Insert the bag from the left side (smoothly and effortlessly).
- The distance from the object to the sealing line must be higher than 2.5cm.
- If you detect any abnormal phenomenon, turn off the sealer and solve the problem before continuing.

9. Maintenance

To clean the machine, first turn off the machine and wipe with a soft cloth.

Make sure that the machine is cool when replacing the blade. Be careful with the blade, it is very sharp.

10. Troubleshooting

1. Paper

If there is an error when loading the paper, check the size of the bags, check that they are correctly placed, check that the distance between bags is more than 1.5 seconds before and after the bag.

2. Print head

If the printhead overheats, an error occurs. The print head temperature has exceeded the maximum set operating temperature. When the head automatically cools, the device will work normally again.


3. The equipment does not boot:

Check if the equipment is connected and if the power supply is normal; check that the power switch; Check if the fuse is damaged. Once these checks have been made, restart the device.

If you still cannot use the equipment correctly, contact the technical service or the manufacturer.

4. Simple troubleshooting:

No.	Failure	Analysis	Solution
1	No power supply	Is the equipment plugged in or open?	Connect it to the electric current
2	Engine does not turn	Is there a paper jam?	Do not feed cards
3	Poor sealing	Adjust temperature	Repair or replace

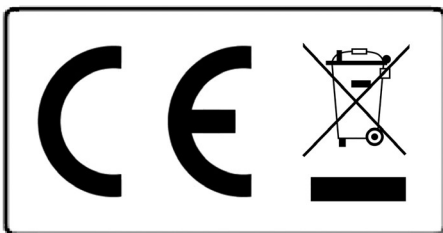Notice to customers:

The product is made up of various components and various materials that must be recycled or, failing that, deposited in the corresponding debris removal sites when the product's life has been completed or when otherwise it is necessary to dispose of it. To do this, the end user who acquires the product must know the current regulations of each municipality and / or locality based on the waste electrical and electronic equipment. The user who acquires this product must be aware of and responsible for the potential effects of the components on the environment and human health as a result of the presence of hazardous substances. Never place the product in a conventional container of citizen scope if a previous dismantling and knowledge of the components that incorporates. If you do not know the procedure to follow, consult with the city council for more information.