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1. General Information


- Handle the parcel with care. Unpack and check that the contents coincide with the packing-list. If any part is damaged or missing, please advise the distributor immediately.
- Do not install or use the equipment without reading this handbook first. This instruction manual must be attached to the equipment and must be available for all users.
- If you have any doubts or enquiries, contact with J.P. SELECTA's technical service.
- Do not use the equipment in explosive, flammable or corrosive atmospheres.

2. Packing List

Model	3001252
Instructions manual	80394
Double-deck tray	0059580
Power cable	0007140

3. Technical Specifications

Model	3001252
Capacity (bags):	from 4 to 6
External dimensions: height/width/depth	34/41/50 cm
Power supply:	See features label
Electrical power (W)	1000
Temperature range (°C):	Fixed 37°C
Display resolution:	1°C
Temperature stability:	±0,5°C
End of run time alarm:	Acoustic and message on screen
Weight:	20 kg.
Last cycle internal memory:	Yes
USB connection:	Yes

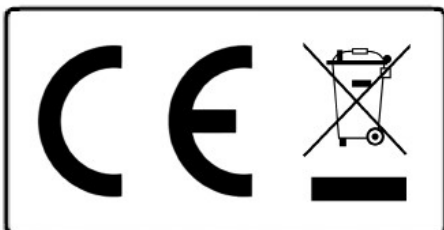Notice to customers:

The product is made up of various components and various materials that must be recycled or, failing that, deposited in the corresponding debris removal sites when the product's life has been completed or when otherwise it is necessary to dispose of it. To do this, the end user who acquires the product must know the current regulations of each municipality and / or locality based on the waste electrical and electronic equipment. The user who acquires this product must be aware of and responsible for the potential effects of the components on the environment and human health as a result of the presence of hazardous substances. Never place the product in a conventional container of citizen scope if a previous dismantling and knowledge of the components that incorporates. If you do not know the procedure to follow, consult with the city council for more information.

4. Equipment Description

This equipment is designed to defrost and heat blood derivatives in bags, before transfusion. It is suitable for plasma, whole blood, concentrated erythrocytes and cryopreserved preparations.

DRYBATH PL-3 dry air defroster allows defrosting of up to 400ml bags through a stream of hot air and a gentle agitation. Bags are placed in perforated trays inside a

5. Installation

Unpack the equipment and visually check that the contents are in good conditions. Keep the package for some days.

5.1 Location

Place the equipment on a flat level stable surface, suitable for the equipment weight.

To work in a comfortable way, please leave a free space of 50cm around the equipment and a minimum of 20cm at the rear part.

5.2 Connections to the mains

Plug the equipment to a near socket suitable for the equipment power. Use the connection cable supplied or a similar one.

For your safety, the equipment must be earthed.

Before plugging the equipment, check that the voltage supply and the frequency corresponds to the one indicated in the equipment characteristics plate.

If you change the plug, bear in mind the following:

Blue cable:	Neutral.
Brown cable:	Phase.
Yellow/Green cable:	Earth.

6. Operation

6.1 Elements identification

1. Access cover to the tray
2. Control screen
3. USB connector

- 4. General switch
- 5. Network connection base + fuse

Removable inner tray to facilitate cleaning.

Double-deck tray for bag placement.

Note: The equipment design is based on placing the bags in a stream of hot air. NEVER use the bath without the trays, as the air will not circulate and the operation will not be adequate.

ATTENTION!! It is imperative to place the plasma bags in position 1.

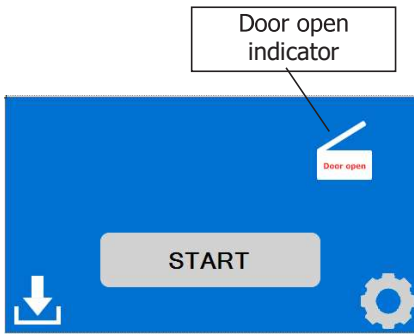
Fig 1. Starting up screen

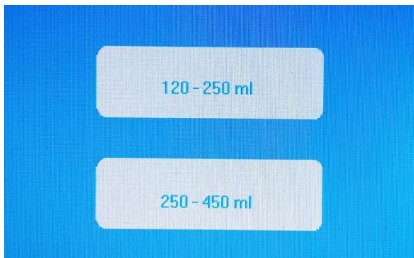
Fig 2. Type of bag selection screen.


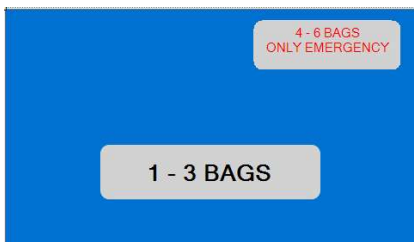
Fig 3. Program selection screen

Fig 4. Program running screen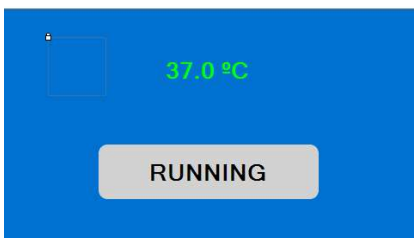

6.2 Starting up

Before starting up, please be sure that:

- You have read the instructions referred to in paragraph 5 «Installation».
- Activate the main switch (4) located at the rear part of the equipment.
- All actions are carried out through the control screen (2).
- When starting the equipment up, the work screen will appear. This screen indicates that the door is open and you will not be able to start the cycle until it is closed. When you open the door, the fan will stop heating and stirring.

An icon also appears to be able to download the recorded data from the previous process.

6.3 Equipment usage

Once the main screen starts up, it turns on and the button for START appears on screen. Then, press the key and a screen will appear where you must select the type of bag you are going to use.

Next, the operating mode must be selected. The equipment has two operating modes:

- The standard functionality is for most "1-3 BAGS" applications.
- The extra operation to place up to 6 bags "4-6 BAGS" (This process is only recommended for exceptional cases of high load and what it does is two heating sequences. At the end of the first phase it will be necessary to remove the bags from the top tray and continue the process with the bottom tray).

- Once the process has started, it cannot be stopped to prevent it from accidentally stopping. It can only be stopped if the equipment is turned off with the main switch (4).

-While the bags show freezing signs, the ice symbol appears and the temperature reading will not be displayed. The reading will only be shown after 25°C.

- At the end of the process it will indicate **END OF PROCESS**.

Note 1:


If the door is opened during the process, the door open symbol will appear flashing in the upper right corner, until the process is complete. There will be evidence that the door has been opened by a user.

Note 2:

Once the process has started, it is not possible to add more bags. Temperature inside the equipment would not be uniform and bags would not defrost properly.

7. Temperatura Register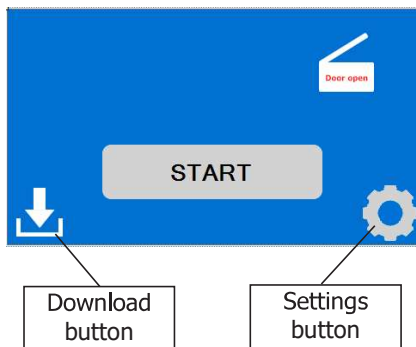


The equipment records the last process carried out. These data can be downloaded to a USB memory for further treatment or storage.

The generated file is type .CSV or .TXT and it is possible to open it with most spreadsheets or word processors.

In order to download the data, you must press the download button on the main screen, but if the process is running, it will not be possible to download this data. The button will only appear if the process is stopped.

Insert the USB memory into the connector (3) and follow the instructions on the screen.

8. Settings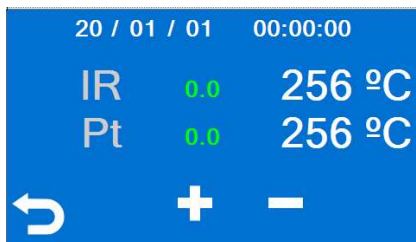

Press the Settings button. The screen to adjust the temperature offset will appear, as well as date and time.

Click on the parameter you want to modify and use the + and - keys to modify the values.

Changes will be saved when exiting this screen.

9. Warranty

This product has a warranty of one year. The warranty does not cover damages caused by an inappropriate use or causes beyond J. P. SELECTA, s.a.u.

Any manipulation of the equipment by not authorized personnel cancels the warranty benefits.

10. Maintenance

Before removing the cover of the equipment, disconnect it from the mains. The control board should only be handled by duly authorized maintenance personnel.

Cleaning:

- Inner tank: clean with a stainless steel rinse aid.
- Outer case: clean with a damp cloth or with a mild detergent.
- Touch screen: use a cloth and a glasses-cleaning liquid.

Annually:

- Contrast temperature against equipment traceable to recognized standards.
- Visually check the condition of the electrical connections, grounding and connection cable to the electrical network.

11. Spare Parts

36027 Solid relay
 39294 Electrical resistors (x2)
 24340 Turbine
 29560 Control board
 29589 Relay module
 43044 PT100 probe
 85167 TFT screen
 28701 Tank swing eccentric
 24000 Swing motor